



A division of
Integrative Veterinary Education, Inc.

ive.

Volume 1, Issue 1

July 2005

Alpha Waves

EXPANDING THE KNOWLEDGE ABOUT ALPHA-STIM FOR ANIMALS

July 29-30, 2005 marks the first Alpha-Stim® training seminar specifically for animals.

There have been many landmarks of expansion for Electromedical Products International, Inc. (EPII) into the field of veterinary medicine over the past few years. A great research study with behavior in horses following Cranial Electrotherapy Stimulation (CES) application was conducted in England, animals were added to the AAPM program in September 2003, and an exhibit and 4 hour wet-lab presentation at the Western States Veterinary Conference in February of this year have all added to this movement into the world of animal therapy.

“Animals are very in tune to their bodies and will generally respond quicker to Alpha-Stim therapy than do humans,” says Dr. Ava Frick who according to Dr. Dan Kirsch of EPII is the world’s leading authority of micro-current therapy in animals.

WHY USE CONDUCTING SOLUTION?

Alpha Conducting Solution is well researched and tested. The ingredients as listed on the MSDS sheet are: 1,2 Propanediol; Purified water; Conducting salts; F D & C Colorants.

This formulation opens the skin to

This month in Union, Missouri Ava Frick, DVM will instruct veterinarians, technicians, and Alpha-Stim® distributors on the theories and application of CES and Microcurrent Electrotherapy (MET). The program is 11 hours; 8 of lecture and 3 of hands-on usage on both dogs and horses and is approved for CE units.

This is a great opportunity to learn about real cases, real results, and how to get the most out of your time and equipment. Added entertainment is scheduled for Friday evening and Saturday afternoon or plan a little extra time in Missouri to visit some of the local wineries.

Early bird registration closes July 15, 2005 so call now to get signed up!

Call either 636-583-1700 or 866-836-3900 or to get fees, lodging, and to register. See you there!

allow the electrical impulses to be conducted under the skin. Saline cannot do this and is not approved for this purpose. While alcohol is used to clear the skin surface of fats or oils, it should not be used as a conducting solution because it is a drying agent.

Upcoming *Alpha Waves* ... Dr. Frick having directed more than 3,000 animal Alpha-Stim® treatments will address specific protocols to customize each species you work with!



TOBY ENJOYS HIS THERAPY

Toby is a rescued basset hound that had a rough start. Following the diagnoses of severe hip arthritis secondary to an earlier injury, he underwent surgery to remove the head of the femur in order to relieve arthritis and pain. But that was not successful and the leg was eventually amputated.

Toby was experiencing a chronic pain syndrome and drug therapy alone was not enough to ease his pain. CES therapy was prescribed and the owners administered it at home on a daily basis for two weeks. At that time Toby’s attitude, appetite, sleep and interaction with the family was back to normal.

“It was the only time he could sleep,” says Nora his mom about the CES treatments she applied at home.”

**A division of
Integrative Veterinary
Education, Inc.**

1825 Denmark Road
Union, MO 63084

Phone: 636-583-6293
Fax: 636-584-0371
E-mail: drfrick@yhti.net



**Making Waves in Medical
Therapy!**

**VETERINARIANS AND CHIROPRACTORS: Plan to attend the Wild
West Veterinary Conference October 5-9 in Reno, Nevada. Dr Ava
Frick will be presenting "Harmonic Resonance of the Autonomic
Nervous System. Balancing it with Chiropractic, Nutrition,
and Microcurrent Therapy."
A 4 hour Wet-Lab is available Friday Oct 8th.**

Taking Care of Your Equipment

Alpha-Stim® units are durable but it is important to take care of your equipment treat them as a medical device. They are easy to service in clinic. Our patients will not tell us if they feel nothing so we need to take these simple steps to make certain they are getting the therapy we think we are delivering!

Remind staff and clients to remove the ear clips by pinching the clips to release them from the ears rather than pulling them off the ears by the connecting line. Check probes and ear clips once a week if you are using them daily to ensure all parts are in working order. For leased units I recommend servicing on each return before being sent out again.

PROBES may be tested by applying the conducting solution, then placing one on each temple and turning the unit on. You will feel the current or see some flashing lights. Test both channels 1 and 2.

EAR CLIPS can be checked by connecting them and enjoying a CES treatment!

Keep the unit clean by using alcohol or a disinfectant on the outer parts. Be careful not to overdo it and get excess liquid inside. When changing the pads, new ones will adhere better if you clean the ear clips or end of probes thoroughly with alcohol and dry them prior to application of a new pad. Another recommendation is to do this at the end of a treatment rather than before initiating a therapy so that the pads have a longer time to set before the conducting solution is applied.

All units have a 5 year warranty so please contact the manufacturer with any concerns. They can help over the phone or guide you in getting further service or replacement should that rare time arise.

When treating animals, the fur is a great wicking device, so reapply conducting solution about every 60 seconds!