



Stifle Pain (Left)

(After doing steps 1-6, repeat steps 1-3 on the opposite leg only once.)


If both sides are painful apply steps 1-6 to the left and right stifle.

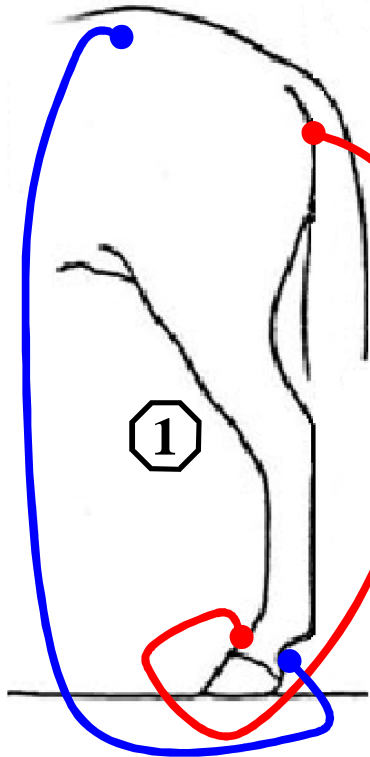



1. Large X: 20 seconds each.

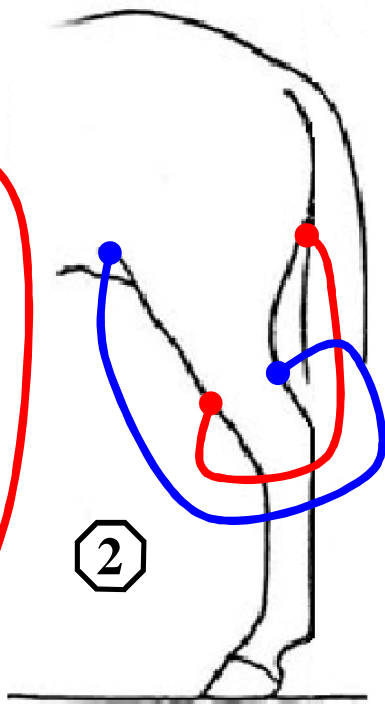


2 & 3. Stars: 10 seconds each. Do multiple stars all directions. Repeat 2-3 times.

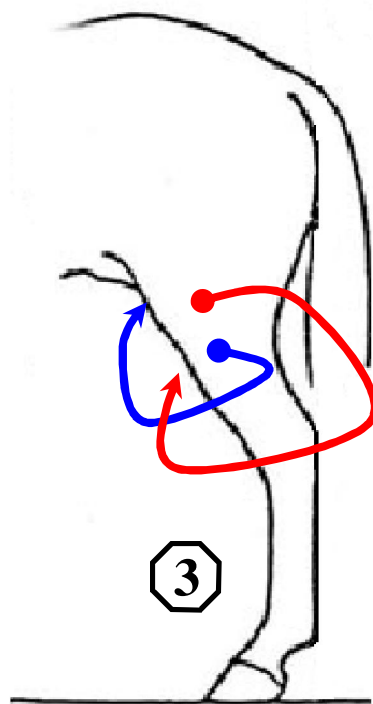


3. One probe inside the stifle (arrows) and one probe on the outside.

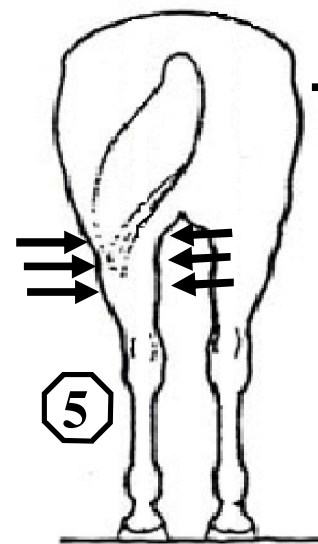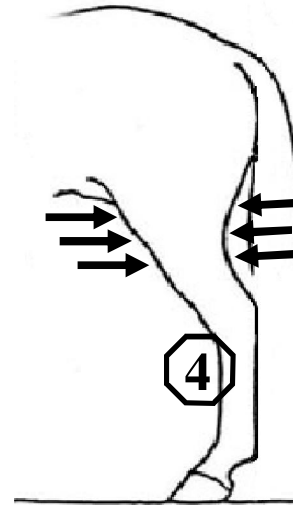

4 & 5. Probes are places directly across each other going through the joint at three planes. 10 seconds each.

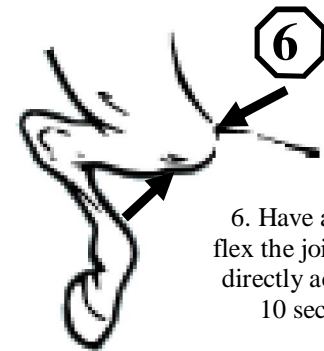

6. Have a helper flex the joint. Probe directly across for 10 seconds.

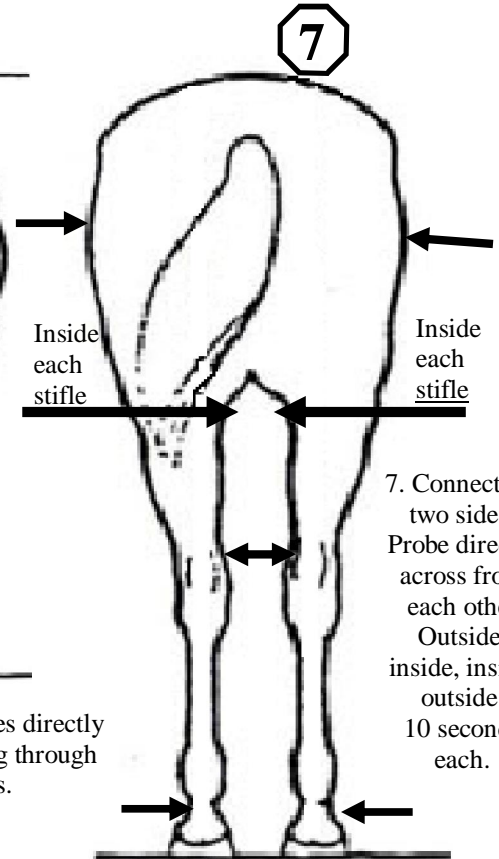

7. Connect the two sides. Probe directly across from each other Outside, inside, inside, outside. 10 seconds each.

8 Then apply ear clips for 20 minutes.